


JTS[®]

TG-iL Induction Loop

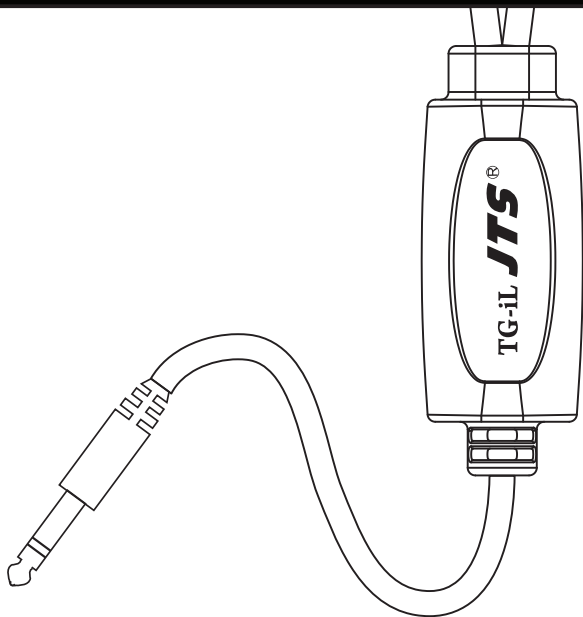


www.jts.com.tw

The TG-iL induction loop is designed for hearing aids. Simply plug the TG-iL into JTS TG-10R/TG-64R, tour guide receiver, MP-3 or mobile phone, the audio signal will be transferred into a magnetic field. A hearing aids with Telecoils (T-coils) will transfer the magnetic field back to audio signal.

1. Dimension:

⌀ 3.5mm Stereo

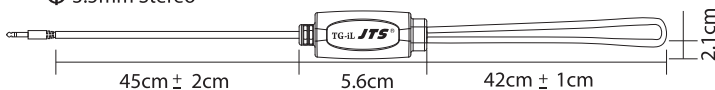

2. Plug type:

3.5 mm diameter, dummy stereo.

3. Operation:

- (1) Plug the TG-iL into a sound source.
- (2) Adjust the volume of the sound source and the hearing aids to a proper level.
- (3) Please note at some position of the loop the sound quality will be better than others. In general the closer to the loop the better the sound quality will be. The best position is indicated by the following figure with dotted line.

